Reprop® Clyster

Starter-Set "Basic"

REF PR101

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Starter Set "Basic"

REF PR101

Components:	1 bottle 750 ml with air release valve
	1 extension tube with check valve
	1 rectal tube

Starter Set "Plus"

REFP R102

Components:	1 bottle 750 ml with air release valve
	1 extension tube with check valve
"Plus":	3 rectal tube
	1 storage bag

Replacement accessories kit

REF PR103

Components:	1 extension tube with check valve
	10 rectal tubes

Directions for use

Mechanism of action: The water introduced via the clyster causes both softening and liquefaction of the feces and stimulation of the rectum via bowel wall expansion. Once the introduced amount of water has reached the reflex threshold, it triggers transport motions, resulting in evacuation of the bowel.

Indications: For fast evacuation of the rectum in cases of constipation, for stimulation of bowel activity in cases of intestinal hypomotility, for control evacuation of the bowels in cases of fecal incontinence. In naturopathy: for bowel cleansing, e.g. when fasting

Further special indication (for health care professionals): For fast evacuation of the rectum preoperatively, prior to rectosigmoidoscopy, prior to pelvic examinations, for controlled evacuation in cases of colostomy.

Enema administration is contraindicated in the presence of acute abdominal and pelvic problems (abdominal pain and vomiting of unknown origin), injuries in the anal region.

Special warnings and precautions: Use clean water only (drinking water quality). Apply lubricant to tip of rectal tube to avoid mechanical irritation of the anal canal. Should resistance or pain be experienced during introduction of the rectal tube, stop procedure instantly and seek medical advice.

Instructions for use

Preparation: Fill bottle with clean water of body temperature, screw on lid, join check valve and rectal tube, as well as extension tube and bottle. Keep gloves, lubricant, water-resistant protective sheet and waste container at hand.

For patient (self-use): Sit down comfortably on the toilet seat or lie down relaxed on your back and gently introduced the rectal tube for a few inches. Turn the bottle upside down and pump the water into the rectum by manually compressing the bottle. Then remove the rectal tube and evacuate your bowels. Repeat process, if required.

For nursing care provider (administration in patients): Preferably, place the patient on his/her left side with the knees flexed. Gently insert the rectal tube. Turn the bottle upside down and allow the water to flow into the rectum. Fluid administration can be easily stopped by simply using the clip. Remove rectal tube. Support patient's evacuation of the bowel, if necessary.

Recommendations for cleaning and reuse: For hygienic reasons we recommend the **exclusive personal (patient-specific) use**, i.e. reuse only by the same person (e.g. for patients at home, in retirement homes or nursing homes). Rectal tube for single use only. Reuse of the other components of the device after cleaning: Manually with soapy water, subsequently dry well. Replace extension tube (incl. check valve) after 10 applications.

Storage information: Store dry and dust-protected at room temperature. Keep out of reach of children.

Further information: www.reprop.ch

Manufacturer: Practomed GmbH, PO Box 290, CH-8042 Zurich (Manufacturer of rectal tubes: MEDICOPLAST, D-66557 Illingen)

In case of queries, seek professional advice.

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Fabrikant: Practomed GmbH, Postfach 290, CH-8042 Zürich